






# Android

1. Telefonā lejupielādējiet *librelink.apk* no [https://drive.google.com/file/d/1UHp64gW-lkntU4SH2\\_iizDU-hn5AVmNG/view?usp=sharing](https://drive.google.com/file/d/1UHp64gW-lkntU4SH2_iizDU-hn5AVmNG/view?usp=sharing)  
Jāspiež *Download* -> *Download anyway* -> *OK*

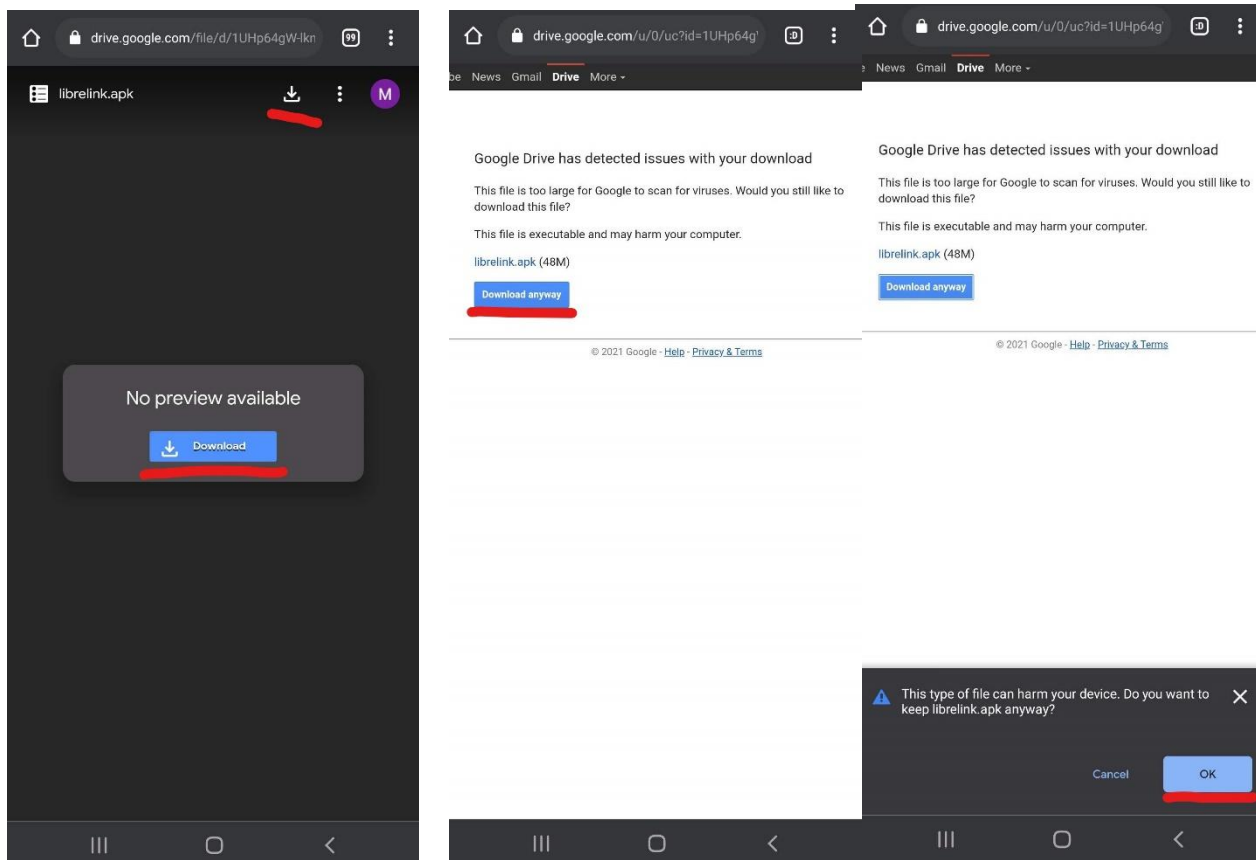

2. Atveriet un palaidiet lejupielādēto failu. Ja līdzīgi faili telefonā iepriekš netika instalēti, būs nepieciešams iedot atļauju to darīt.

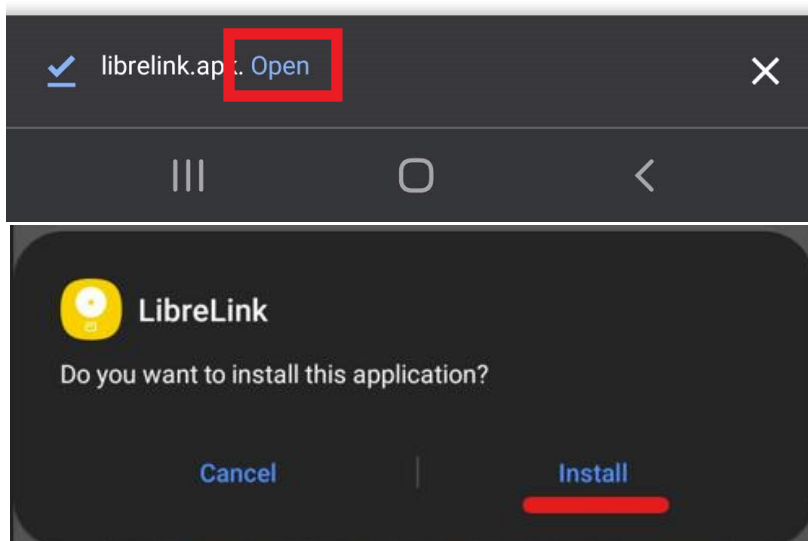

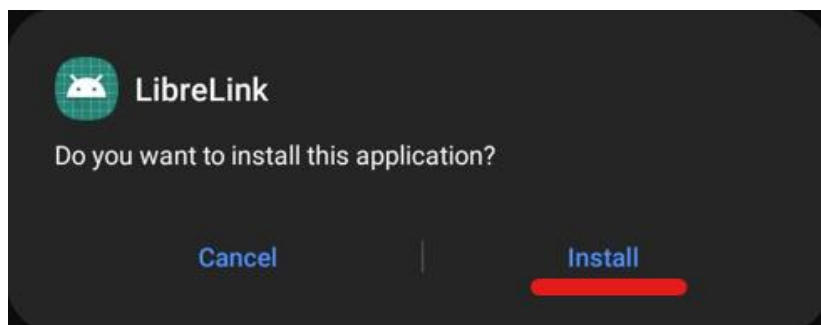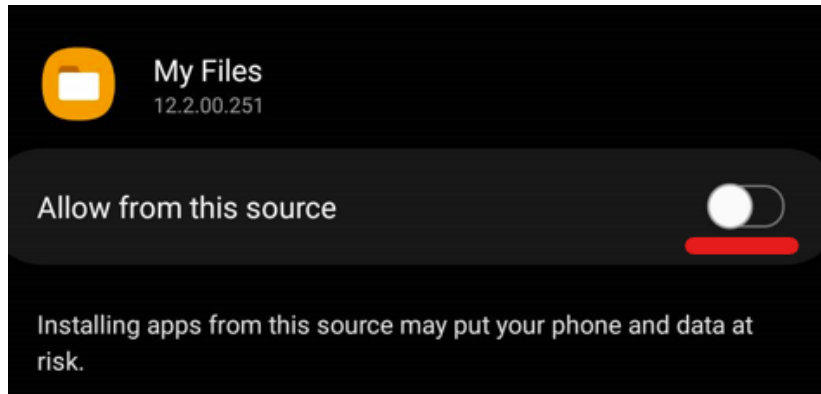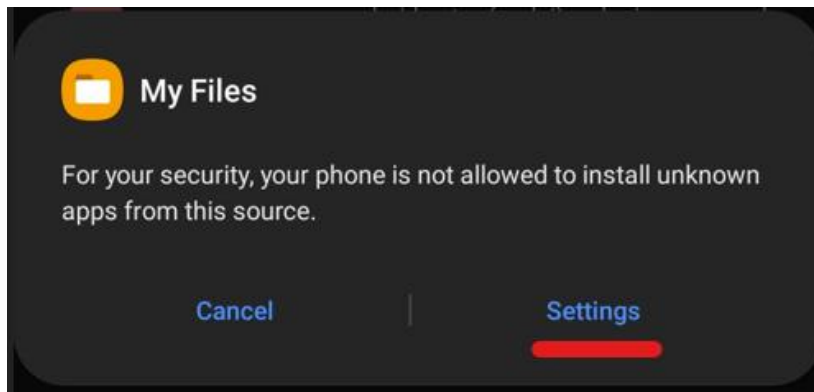

3. Pēc instalācijas atveriet aplikāciju un norādiet sekojošos iestatījumus:


1) mmol/L

Unit of Measurement


Select the unit of measurement for your glucose. All glucose data will be displayed using these units. You will NOT be able to change this later.

☐ mg/dL (for example 100 mg/dL)


☒ mmol/L (for example 5.6 mmol/L)

 It is VERY important to select the right unit of measurement for your glucose. The wrong unit of measurement may lead to misinterpretation of your glucose values. Check with your health care professional if you do not know which unit of measurement to use.


2) grams


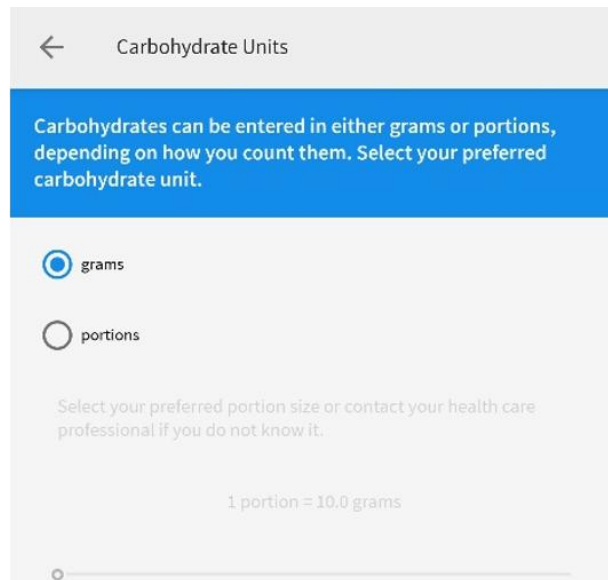


### 3) sound and vibration

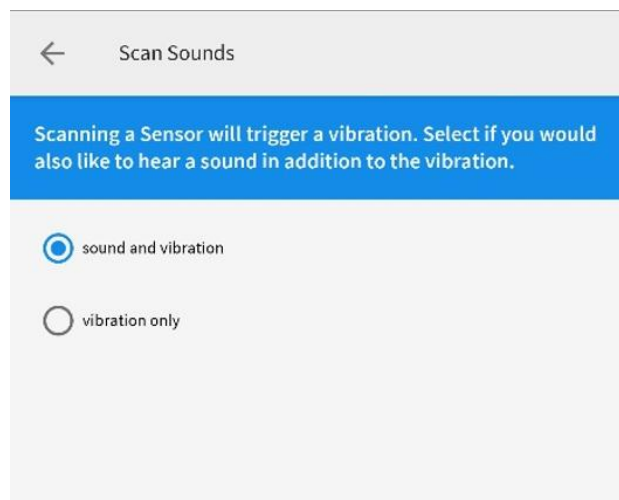

4. Tagad LibreLink aplikācijai nepieciešams iedot piekļuvi lokācijai un failu glabāšanai uz ierīces. Jāiet iestatījumos -> Apps -> LibreLink -> Permissions -> Location/Storage

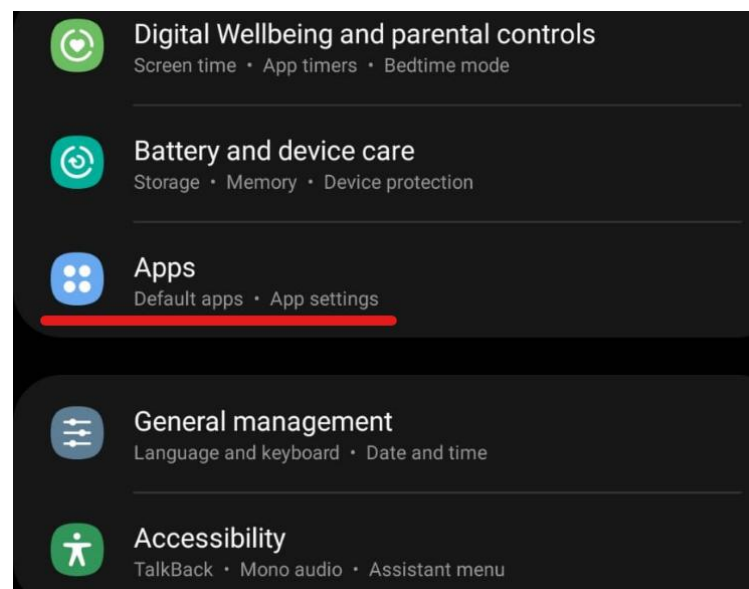

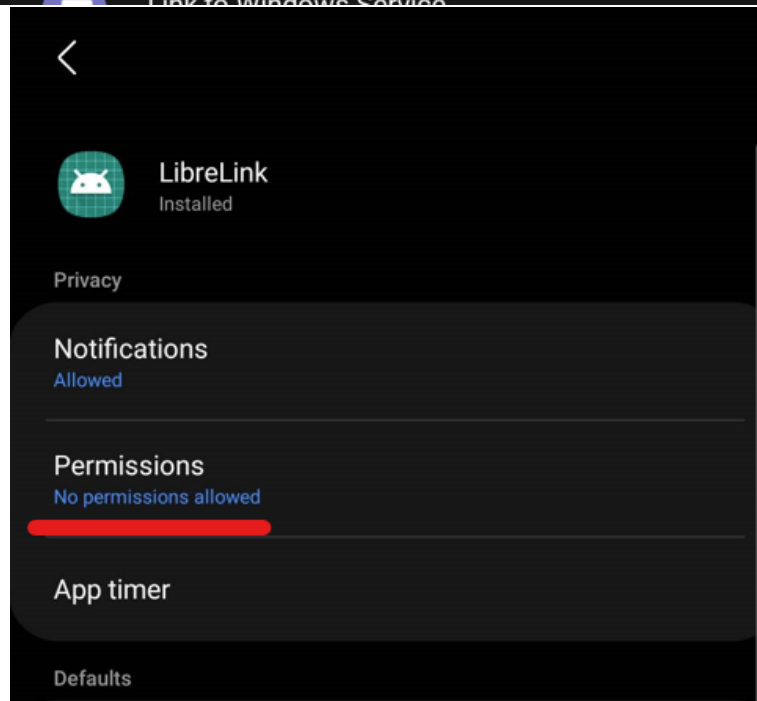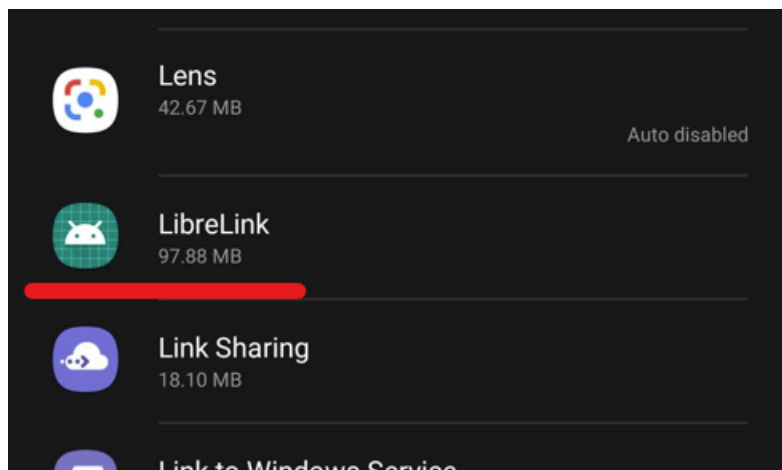



LibreLink

ALLOWED

No permissions allowed

DENIED



Files and media



Location



Location permission



LibreLink

LOCATION ACCESS FOR THIS APP



Allow all the time



Allow only while using the app



Ask every time



Deny



## Files and media permission



LibreLink

### FILES AND MEDIA ACCESS FOR THIS APP



Allow management of all files



Deny